


INSTRUCTIONS

SMALL POWER INJECTOR GUN:

1. Remove the plunger from a Monoject 10cc oral medication, 12cc luer lock, or 20cc luer lock syringe. Remove the black rubber tip from the syringe plunger
2. Examine the plunger of the Power Injector gun. Locate the washer that can be moved forward and backward. For the 10cc and 12cc syringes, move the washer backward (toward the handle of the Power Injector). For the 20cc syringe, move the plunger forward (away from the handle of the Power Injector).
3. Press the plunger release on the top of the Power Injector and slide the Power Injector plunger as far toward the handle as possible.
4. Place the black rubber tip on the tip of the Power Injector plunger. When using the 10cc or 12cc syringes, the rubber tip will not fit over the washer. When using the 20cc syringe, the rubber tip must fit over the washer.
5. Place the barrel of the syringe over the Power Injector plunger, which has the black rubber tip attached. For the 10cc, and 12cc syringes, push the collar of the syringe barrel into the deepest recess of the barrel holder until it snaps into place. Turn it 90 degrees to lock it into place. For the 20cc syringe, place the collar of the syringe into the shallow larger recess of the barrel holder until it snaps into place. Turn the barrel 90 degrees to lock.

The Tulip Power Injector is autoclavable. Do not clean with *alkaline* based cleaners

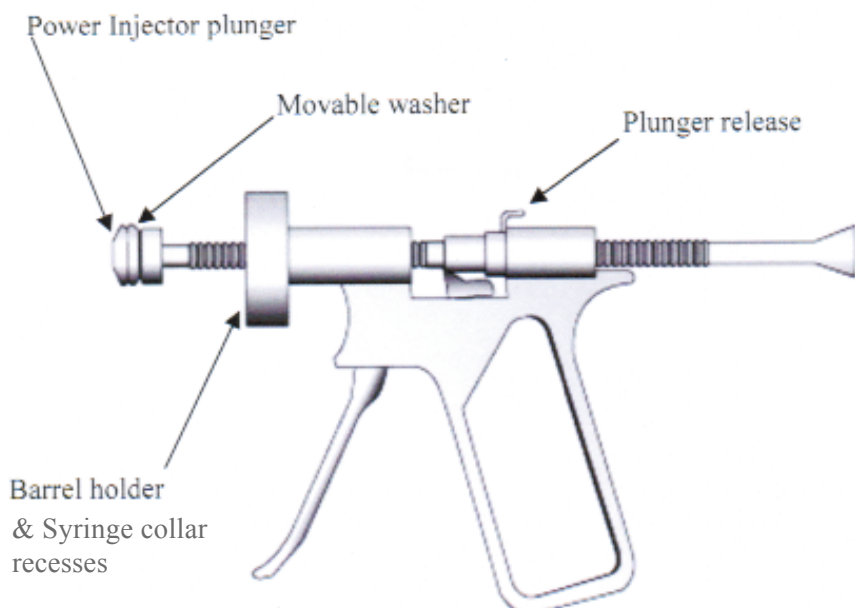

Power Injector Gun

The Power Injector allows surgeons to reinject soft tissue through large and small diameter injectors with very little effort and maximum control. The Small Tulip Power Injector will accept any Monoject 10cc oral medication, 12cc luer lock, or 20cc. luer lock syringe.

Note:

These steps must be performed in a sterile field.