



Tulip Decant Stand Instructions

The Decant Stand works with any size syringe that has a luer lock hub. Autoclave the Decant Stand.

Harvest the tissue then attach the full syringe to the small end of the Decant Stand by screwing it on. Place the stand upright on the sterile field. In a minute or two the fat will separate (float) from the liquid. Expel the liquid through the Decant Stand. Attach the injection syringe of choice to the luer lock attachment in the bottom, wide end of the Decant Stand and draw off the fat. Continue to load injection syringes as desired. Discard any oil or lipid at the top of the decanted fat.

