

Tulip[®]

**TULIP ANAEROBIC
TRANSFER**

DECANT STAND™

Quick Start Guide

800.325.6526 | sales@tulipmedical.com





Tulip Anaerobic Transfer Decant Stand TM Instructions

Anaerobic transfers are used to prepare and transfer extracted tissue for fat re-injection. Transfers incorporating the Decant Stand feature can also act as a syringe stand keeping your syringe upright allowing for a gentle, natural gravity decant. Decanting and transfer should take place in the sterile field.

Please read package inserts prior to use.

- 1 After harvesting tissue, attach the transfer to the harvesting syringe and stand the transfer on its base. Gravity will cause the tissue and fluid to separate.
- 2 Expel the fluid for disposal.
- 3 The decanted tissue is ready to transfer.
- 4 To fill an injection syringe, attach it to the connection in the base of the stand.
- 5 Draw off the desired amount of tissue into the injection syringe by gently pulling on the plunger.
- 6 Repeat steps 4 through 5 to create additional injection syringes.

